

SURGICAL TECHNIQUE

VersaPEEK™



Product Overview

VersaPEEK™

The VersaPEEK™ is a PEEK suture anchor used in Rotator Cuff Repair, general soft tissue fixation, and complex fracture surgeries. The anchor accommodates a wide variety of suture or suture tape configurations and is engineered to provide ultimate pull-out strength.

Features and Benefits

- Minimal post-operative imaging effects
- Engineered for enhanced mechanical properties (pull-out strength, tensile strength, etc.)
- Variety of suture/suture tape configurations
- Open-handle design - Needles are retained inside the handle

1 PREPARE THE ROTATOR CUFF FOOTPRINT

Preparation of the greater tuberosity.





2 PRE TAP HOLE

Identify the medial anchorage site and create a pilot hole using the VersaPEEK™ tapping tool. It is important to always create a pre-tapped hole as the VersaPEEK™ does not have a self-cutting tip.




3 INCISION AND EXPOSURE

Make sure the anchor tip is placed appropriately into the pre-tapped hole on the medial cartilage edge. The VersaPEEK™ is inserted with an accessory portal. The anchor tip is inserted in the pre-tapped hole until the thread is engaged. Screw the anchor into the bone until the anchor sits flush with the cortex.

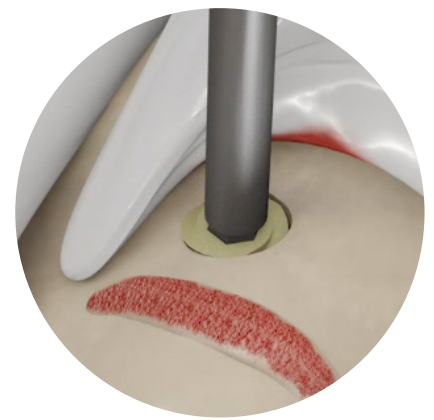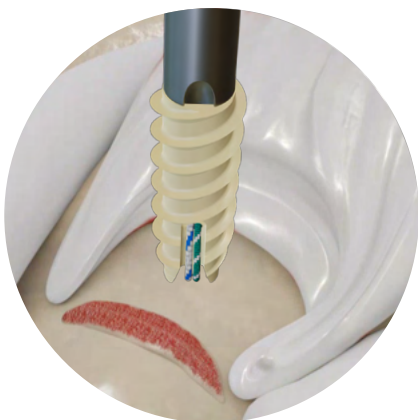

4 INSERTER REMOVAL

Release the sutures at the back of the handle and remove the inserter.

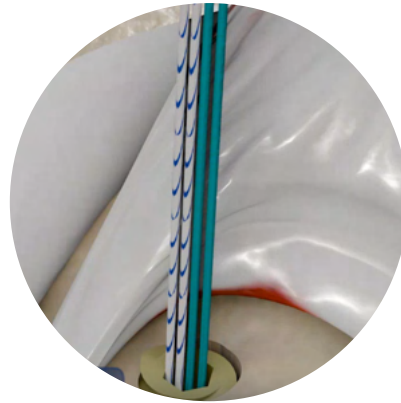

5 SUTURE PASSING

The sutures of the VersaPEEK™ are passed through the indicated soft tissue, in the desired configuration as indicated. There are multiple sutures or suture tapes available. These sutures can be used for reduction of the cuff or to attach any soft tissue to bone using the preferred method.



References

ADP001	VersaPEEK™, PEEK Suture Anchor, 5.5 x 15mm
APP001-N	VersaPEEK™, PEEK Suture Anchor, 5.5 x 15mm, w/needles
APP002	VersaPEEK™, PEEK Suture Anchor, 4.5 x 15mm
APP002-N	VersaPEEK™, PEEK Suture Anchor, 4.5 x 15mm, w/needles